



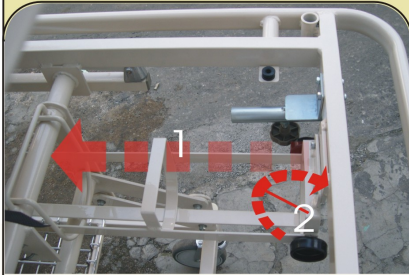

OXYGEN BOX CONTAINER

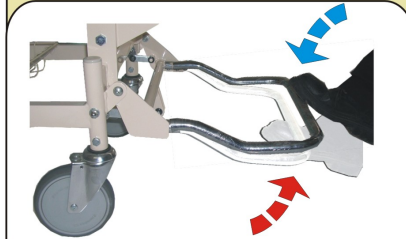


1. Put the oxygen box at the container
2. Lock the containe

TOTAL LOCK




-  Step on the wheel lever to lock the wheel.
-  Lift up the wheel lever to release.

IRRIGATOR BAR BRACKET

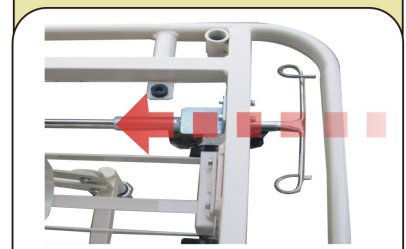

Put the Irrigator Bar on the bracket (at frame head side) if notbeing used.

FOLDING SIDERAIL

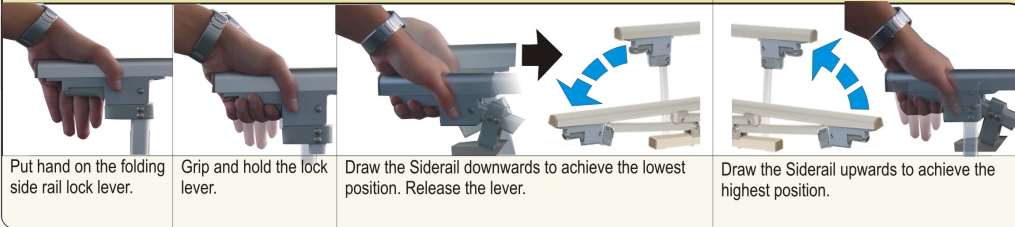

Put hand on the folding side rail lock lever.

Grip and hold the lock lever.

Draw the Siderail downwards to achieve the lowest position. Release the lever.

Draw the Siderail upwards to achieve the highest position.

Customer Service Contact :
PT Chitose Indonesia Mfg
 Jl. Industri III No. 5 Leuwigajah
 Cimahi 40533 - INDONESIA
 Phone : (022) 603 1900
 Fax : (022) 603 1855
 e-mail : aan@chitose-indonesia.com

Chitose[®]
 Innovation by your Inspiration