



CAUTION LABEL



SAFETY NOTICE

- Do not use the bed under abnormal condition.
- There is only one person to be allowed on the bed when crank handle is turning to raise or to lower the high level of the bed (crank operation).
- Make sure that patient's body is not under the bed or under/ in the middle of the side rails when crank is in operation.
- The side rails must be properly installed when a patient is sleeping.
- The crank handle must be located to the safe/ locked position by folding it inside after it has been use.
- The crank handle must be always on safe/ locked position to avoid accident leg or hits.


DO NOT REMOVE THIS LABEL

PT.Chitose Indonesia Mfg.,  
Tlp. (022) 6031900 . Fax (022) 6031855



CAUTION !

- Do not place your hand under the bed
- Maximum capacity is one person only

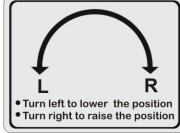


CAUTION !  
CRANK HANDLE OPERATION



- Before turning the crank handle please be aware with hand, children, pets, and other material.
- After turning the crank handle, put it back on locked position in order to avoid accident by sudden falling down.
- In order to avoid leg hit, told the crank handle back to the position (locked). If hit, the bed might fall down.

MANUAL GUIDE  
NURSING BED

CB-135D

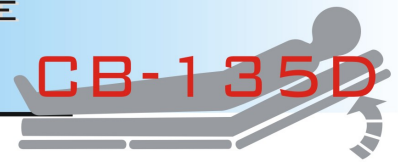

CONTENTS

Topic	Page
Overview	1
Crank Operation Guide	2
Foot Board And Head Board Installation Guide	3
Back Rest/ Bottom A Operation Guide	4
Foot Rest/ Bottom C Operation Guide	5
Side Rail Installation Guide	6
Caster Operation Guide	6
Important Notice Bed Operation	7




Production by :  
**Chitose**



Customer Service Contact :

**Chitose**

PT. CHITOSE INDONESIA MFG.  
Jl. Industri III No. 5 Leuwigajah  
CIMAHI 40533 - INDONESIA  
Phone : (022)-6031900  
Fax : (022)- 6031855

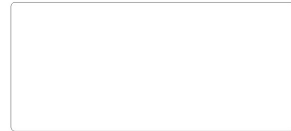

website : [www.Chitose-indonesia.com](http://www.Chitose-indonesia.com)  
e-mail : [Marketing@chitose-indonesia.com](mailto:Marketing@chitose-indonesia.com)

website : [www.Chitose-indonesia.com](http://www.Chitose-indonesia.com)  
e-mail : [Marketing@chitose-indonesia.com](mailto:Marketing@chitose-indonesia.com)