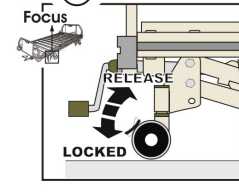
**Indonesia**  
Masukan kedua kaki side rail pada lubang yang terdapat pada sisi kanan dan kiri main frame bed.

**English**  
Insert both of leg of the side rail into their pair holes located at the right and left side of the bed mainframe.

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**English**  
If the side rail is not being used, lift the rail side up from its normal position and remove it to the foot section of the bed mainframe. Insert the both into their reserved pair holes.

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**English**  
Use the caster lock pedal with this directions : Push the pedal down to lock the caster so that it could not be moved. Pull the pedal up to unlock the caster and it can be moved freely.

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**English**  
**USER MANUAL FOR NURSING BED**  
1. Caster  
• Make sure that the casters are unlocked when the bed is moved or removed.  
• Make sure that the caster are locked when the bed is not going to be moved.  
2. Make sure that patient's body is on the safe area before crank and back/ foot rest operations. Inform the patient gently and do the operations carefully.  
3. Head - Foot Board  
• Make sure that Head-Foot Board are always locked.  
4. Side Rail  
• Make sure that both side rails are properly installed, especially when there is a patient on the bed despite being moving or not to moving.