



## SET HEAD AND FOOT BOARD

Focus



Angkat head/ foot board dan masukan kedalam rel pada sisi ujung main frame.



Joint the head/ foot board with the bed mainframe by inserting the hook stick of the bed main frame into the socket groove of the head/ foot board until they are fitting correctly.



Pastikan bahwa head/ foot board sudah dalam posisi terkunci dengan menekan tombol pengunci.



Push down the lock handle to lock the joint of the head/ foot board with the bed main frame.



Tarik ring plastik putih keluar dilanjutkan tarik tombol pengunci ke atas.



Pull the white ring plastic outwards and then pull the key handle upwards.



Angkat head/foot board ke atas untuk melepaskan head/ foot board dari main frame bed.




Lift the head/ foot board up vertically to disjoin the head/ foot board with the bed main frame.



## SET HIGH AND LOW BACKREST

Focus

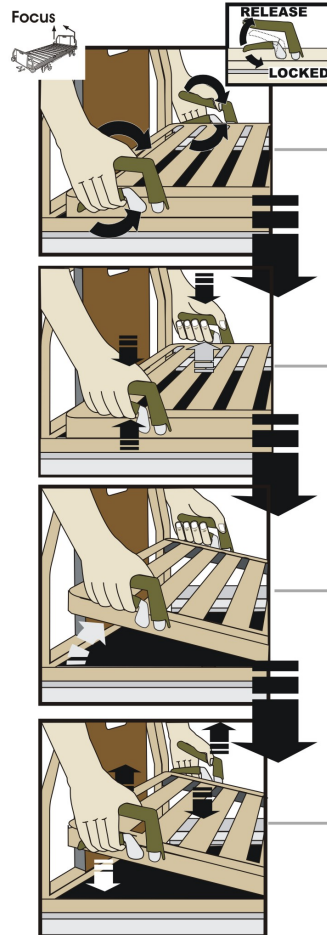

Pergunakan satu/ dua tangan untuk menggenggam handle grip.



Grips the handle grips by one or both hand.



Genggam handle grip di ikuti dengan menarik tuas pengendali. Dan jangan dilepas.



While keep gripping them strongly, pull the control lock handle and hold up.



Pertahankan posisi genggam dan gerakan back rest ke arah posisi yang diinginkan (max. 80).



Move the back rest frame by pulling it up-down slowly.



Lakukan hal tersebut secara benar untuk menaikkan atau menurunkan posisi back rest/ bottom A.



Free your gripping until the comfortable high level has been reached (the maximum level is about 80).