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NURSING BED
Healthy Therapy Product


NURSING BED CB 3300 Plus

Nursing Bed CB 3300 Plus Series is a type of bed that functions to support special needs for people who are sick, disabled, and people with special needs (breastfeeding and the elderly). With the Motor Actuator Operating System as a manual driver for bed height, head and foot raise, it is controlled via a Handheld controller.


Standard Function

- Operating System : Motor Actuator for High-Low, Foot Rest, and Head Rest positions
- Position : - High - low, Automatic operation via handheld Controller with Motor Actuator
 - Head & Foot rest, Automatic operation via handheld controller with Motor actuator
- Primary : - Simple design, no sharp corners (smooth surface)
 - Easy operating mechanism, with Motor Actuator
 - Sturdy and anti-rust
 - Wheels that are easy to move and position, with lock - unlock system
- Lock System : Total Lock System
- Accessories : IV Pole (2 Type Option) and Side Rail / Folding Side rail
 Bed Side Cabinet, Over Bed Table, and Foot Step
 Airmate Pro Mattress / Urethane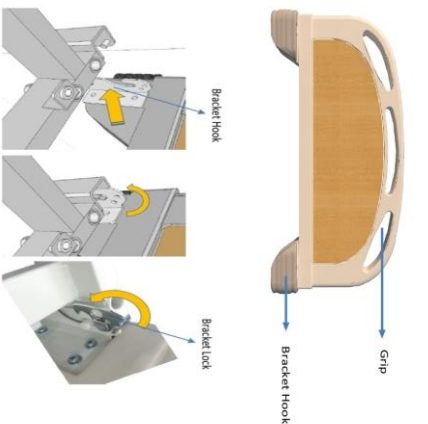


Product Dimension

- Dimension : L : Total 2230 mm (include H/F Board)
 1956 mm (Bottom assy Length)
 W : Total 1030 mm
 915 mm (Bottom assy Width)
 D : 760 ~ 1005 mm (from floor to Head Board)
 : 327 ~ 667 mm (from floor to Bottom Assy)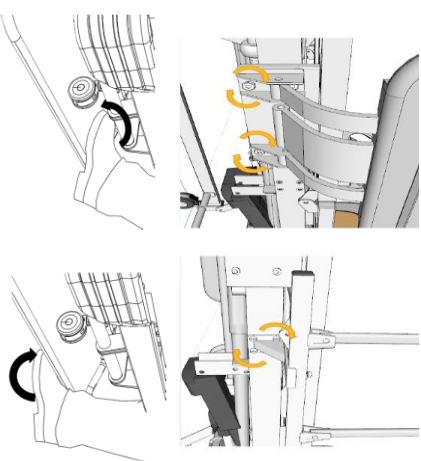

Product Weight

- Weight : ± 90KG
 Load Capacity : ± 400KG (Static) & ±170KG (Dynamic)
 Packing Dimension : 2050 mm (L) x 1070 mm (W) x 190 mm (D)

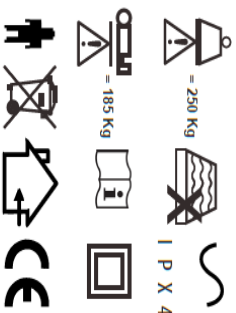AC 230V~50/60Hz
 Duty cycle 2 min ON/18 min OFF
 Int. : 10%, Max 2 min/ 18 min.

INTENDED USE AND PRODUCT DESCRIPTION CB 3300 PLUS

NO	PENGUNAAN	DESKRIPSI PRODUK/ KOMPONEN
1	Designed as inpatient beds in medical institutions, such as hospitals, health centers, clinics, etc., also used for elderly care facilities in nursing homes.	Equipped with an H/F Board on the Main Frame which is installed by attaching the bracket hook on the H/F board to the Stopper Pin, then make sure it is locked with the stopper lock
2	Used for adult patients and not intended for infants or children	Equipped with a mattress that is installed manually by placing the mattress on the surface of the bed. Size and type of mattress according to manufacturer's recommendations. An inappropriate mattress can be dangerous.
3	The bed should not be used for two or more people, as this will exceed the maximum load of the bed	Equipped with an actuator motor that works automatically using an electrical system, which functions to move the headrest (High-Low, Foot Rest, Head Rest) to a comfortable sitting position as well as the shock position and heart position which are operated by handheld controller.
4	The bed construction is built with a lying surface, bottom surface and accessories and is moved automatically with a Motor Actuator system which is controlled using a handheld controller	Equipped with a caster that uses a foot lever to lock and unlock the caster which is manually driven.
5	The total lock castor functions to provide convenience, stability and safety of locking the caster when the bed is parked.	Equipped with side rails as safety on the left and right sides of the bed.
6	It is easy to move because the bed is equipped with castor wheels which make it easier to move	

NURSING BED
Healthy Therapy Product


NURSING BED CB 3300 Plus

Nursing Bed CB 3300 Plus Series is a type of bed that functions to support special needs for people who are sick, disabled, and people with special needs (breastfeeding and the elderly). With the Motor Actuator Operating System as a manual driver for bed height, head and foot raise, it is controlled via a Handheld controller.


Standard Function

- Operating System : Motor Actuator for High-Low, Foot Rest, and Head Rest positions
- Position : - High - low, Automatic operation via handheld Controller with Motor Actuator
 - Head & Foot rest, Automatic operation via handheld controller with Motor actuator
- Primary : - Simple design, no sharp corners (smooth surface)
 - Easy operating mechanism, with Motor Actuator
 - Sturdy and anti-rust
 - Wheels that are easy to move and position, with lock - unlock system
- Lock System : Total Lock System
- Accessories : IV Pole (2 Type Option) and Side Rail / Folding Side rail
 Bed Side Cabinet, Over Bed Table, and Foot Step
 Airmate Pro Mattress / Urethane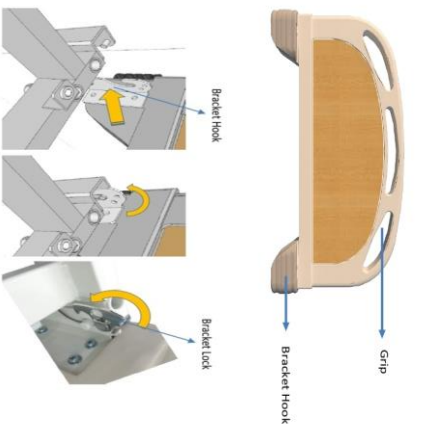


Product Dimension

- Dimension : L : Total 2230 mm (include H/F Board)
 1956 mm (Bottom assy Length)
 W : Total 1030 mm
 915 mm (Bottom assy Width)
 D : 760 ~ 1005 mm (from floor to Head Board)
 : 327 ~ 667 mm (from floor to Bottom Assy)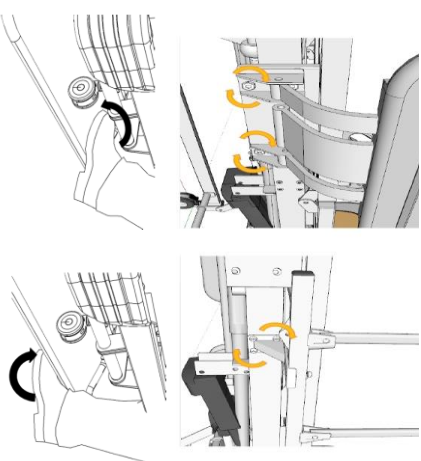

Product Weight

- Weight : ± 90KG
 Load Capacity : ± 400KG (Static) & ±170KG (Dynamic)
 Packing Dimension : 2050 mm (L) x 1070 mm (W) x 190 mm (D)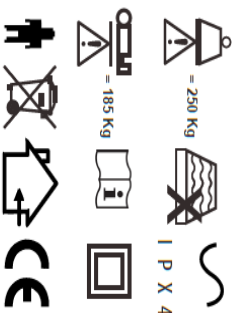

AC 230V~50/60Hz
 Duty cycle 2 min ON/18 min OFF
 Int. : 10%, Max 2 min/ 18 min.