

PETUNJUK PENGGUNAAN, MATERI PELATIHAN, PETUNJUK PEMASANGAN SERTA PEMELIHARAAN CB 3003 D

No	TYPE	PENGGUNAAN	PEMASANGAN	PEMELIHARAAN	SPESIFIKASI KEMASAN
1	CB 3003 D	Designed to be used as an inpatient bed, in medical institutions such as hospitals, health centers, clinics, etc., also used for care facilities for the elderly in nursing homes	Head board and foot board accessories above the head and under the foot of the bed are installed using brackets and lock pins as locks (pictured)	The head board and footboard are checked periodically to ensure that the brackets and pins are still functioning properly	CB/F K200/M150x3/K200 975 x 2350 x 170
		Adjustment system with 3 cranks that are used to adjust the high position of the bed (adjustable Hi-Lo), adjust the high and low position of the patient's back (Back raise) and adjust the height of the knee bend (foot Raise)	Rotate the Hi-Lo crank to the right to raise the bed position and to the left to lower it, Rotate the crank back rise to the right to increase the backrest tilt angle and to the left to lower the backrest angle and crank foot raise to the right to increase the height angle and to the left to lower the elevation angle	1. Beds should be cleaned and disinfected weekly and/or before patients use the bed 2. The headboard, feet and platform mattress pads must be removed from the bed for cleaning	
		The wheel lock system uses a Double lock caster system or one caster lock at the front and one caster lock at the rear to ensure security when in a parking position	Lock and free operation on the caster is done manually by pressing the handle on the caster using the foot	After bed cleaning wipe all bed surfaces with natrium dikloroisosianuat (NADCC) at a concentration of 1000 parts per million (0.1%) of available chlorine	
		It is used for adult patients and is not intended for infants or children under five	Side rails as a safety on the left and right sides of the bed are installed in accordance with the holes provided	The smoothness of the crank rotation is checked regularly whether the function is still good, smooth, not heavy and not noisy	
		The bed may not be used for two or more people, as it will exceed the maximum load of the bed			

OPTIONS



FEATURES & PROCEDURES



Head/foot board panel dismantle



Head/foot board panel installation

OPERATION SYSTEM

A unique eye catching foldable crank is designed to give high-low bed adjustment.




Crank operation system

Side rail compartment
(available in every
nursing bed)

CASTER OPERATION SYSTEM

Independent lock caster (one at front and one at back of the bed) provides stability and secure parking.

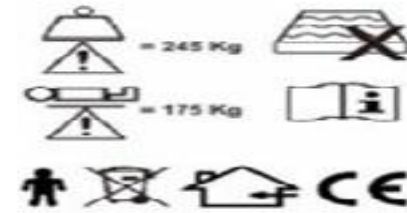

PRODUCT SPECIFICATIONS

Operating System Adjustable Hi-Lo, back raise, and foot raise manual by crank

Accessories Side Rail (CSR-001, CSR-002, CSR 003) & Folding Side Rail (FSR-001)

Lock System Double Lock System

Weight (kg) Approx. 90 / **Frame Finishing** Powder Coating

Frame Colour Ivory / **Frame Material** Steel

Dimension (mm) Bed [W 932, L 2141, H 654-994, H/F Board [W 932]

PRODUCT DIMENSION

