

| TYPE         | USE   | INSTALLATION   | MAINTENANCE   |
|--------------|---|--|---|
| CB 3012 Plus | Designed for use as a patient bed in health institutions such as hospitals or for private patient care  | Prepare the Head/Foot Board on position (Head at the back (patient's head area) and Foot the front (patient's foot area) on the board with the hook found on the main frame, make sure the Bracket Lock is locked  | 1. Beds must be cleaned and disinfected every week and/or before a new patient uses the bed<br>2. The mattress must be removed from the bed for cleaning  |
|              | Functions as a special needs support for people who are sick, disabled, and people with special needs (breastfeeding and the elderly).  | Mattress installation is done manually using a mattress recommended by the manufacturer  | After the bed is cleaned wipe all bed surfaces with sodium dichloroisocyanate (NADCC) at a concentration of 1000 parts per million (0.1%) of available chlorine   |
|              | The manual operating system works manually using the Crank system, to adjust bed height (High-Low) and in a comfortable sitting position and shock position or heart position (head rest)         | There are 2 cranks which have their own functions regarding setting the bottom frame position. When the handle is moved clockwise, the crank will rotate and control the iron pipe torque  | The crank handle must be checked periodically by turning it left and right and adding grease if it is not normal  |
|              | Equipped with other tools such as handrails (fix/loose side-rail & Folding Side rail) on the side of the bed to prevent falls, or Caster (Lock unlock) to facilitate constant movement of the bed | Locking the castor is done manually by pressing the handle on the caster using the foot  | Check the smooth installation and removal of side rails   |
|              | The bed is used in a position that can be moved because the bed is equipped with castor wheels which function to make it easier to move   | Prepare the Side Rail and Position it exactly on the insert provided on the Side Frame, Install the Bolt and Ring on the Insert provided, and tighten it using Lock L (L key) or Screwdriver (+) up to the bolt tightening limit. To raise the side rail, pull the side rail lock, then release the lock and let the side rail move by itself. To fold the siderile, press the button at the end, move the siderile forward while pressing the button and release the button when the position is folded | Perform visual checks on the condition of the castor regularly to ensure the castor is still functioning properly<br><br>Check the mattress for damage and the possibility of water entering the mattress |






NURSING BED


## Healthy Therapy Product



Fitur Head/Foot Board

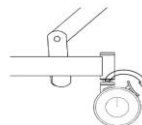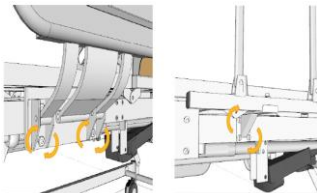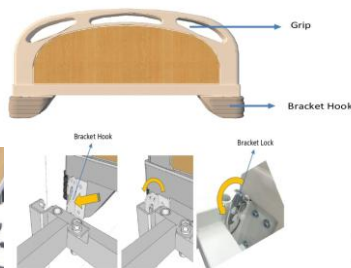

Posilock caster

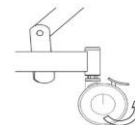

Posilock Unlock Caster

### Standard Function

- Operating System : 2 Crank operation for High-Low and Head Rest positions  
 Position : - High - low: Manual operation with Crank  
 - Head rest: Manual operation with Crank with tilting from 0 to 80 degrees  
 Primary : - Simple design, no sharp corners (smooth surface)  
 - Easy operating mechanism, with Crank System  
 - Sturdy and anti-rust  
 - Wheels that are easy to move and position, with lock - unlock system  
 Lock System : Double Lock System on the Wheels  
 Accessories : IV Pole (2 Type Option) and Side Rail / Folding Side rail  
 Bed Side Cabinet, Over Bed Table, and Foot Step  
 Airmate Cpro Mattress / Urethane

### Product Dimension

Dimension : L: Total 2230 mm (include H/F Board)  
 1956 mm (Bottom assy Length)  
 W: Total 1030 mm  
 915 mm (Bottom assy Width)  
 D : 760 ~ 1005 mm (from floor to Head Board)  
 : 327 ~ 667 mm (from floor to Bottom Assy)

### Product Weight

Weight : ± 90KG  
 Load Capacity : ± 400KG (Static) & ± 170KG (Dynamic)  
 Packing Dimension : 2050 mm (L) x 1070 mm (W) x 190 mm (D)

