MAINTENANCE TRAINING MATERIAL FOR BED TYPE CB 7003 D
Table 1. Crank Assy Replacement Method and Work Sequence * Hi-Lo Case


No	Working Order	Method
1	Raise the Bed to the highest level	1. Raise the Bed to the highest level 2. Turn the Crank Handle to the right and stop where the upstroke becomes heavier.
2	Remove both springs	Remove both Springs using the Hook JIG
3	Lower the Bed to the Lowest level	Turn the Crank Handle to the left and stop where the stoper meets
4	Remove the Comple Crank Rod End	Release the Snap pin and pull the pin
5	Remove the Crank Assy	1. Open the Instruction Label on the handle and open the screw, then remove the handle.
		2. Remove the two M10 bolts attached to the cross member.
		3. Remove the Crank Assy
6	Installation of Crank Assy	1. Install the handle to the cross frame foot side hole and tigh the screw securely
		2. Install the two M10 bolts and tighten
		3. Install the end of the rod axis on the torque tube bracket using the pin, collar and snap pin.
7	Raise the Bed to the highest level	Turn the Crank Handle to the right and stop where the upstroke becomes heavier.
8	Install both Springs	Install the two springs using the hook JIG
9	Lower the Bed to the Lowest level	Turn the Crank Handle to the left and stop where the stoper meets





MAINTENANCE TRAINING MATERIAL FOR BED TYPE CB 7003 D

Table 1. Caster Replacement Method and Work Sequence


No	Urutan Kerja	Metoda/Cara Kerja
1	Prepare everything related to replacement work	Raise the bed by turning the crank handle clockwise until there is enough space (between the floor and casters), then install the stand.
2	Remove the Casters	Loosen the Nut with a 21 mm wrench and remove the caster
3	Install Casters	Insert the caster thread into the foot pipe thread and tighten using a wrench until tigh
4	Make sure the installation is correct	Make sure the 16 diameter Spring Washer is even and has no gaps
5	Take out the chockstand	Turn the crank handle clockwise once there is space then pull the support holder out.
6	Make sure all functions are operational	Make sure the forward and backward rotation is horizontal without any abnormalities. Make sure there are no abnormalities in the lock function.

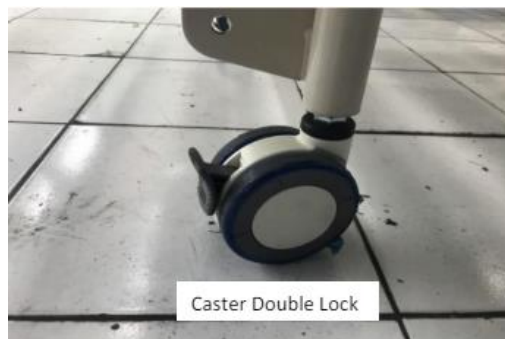
Cara pemasangan Foot Board & Head Board

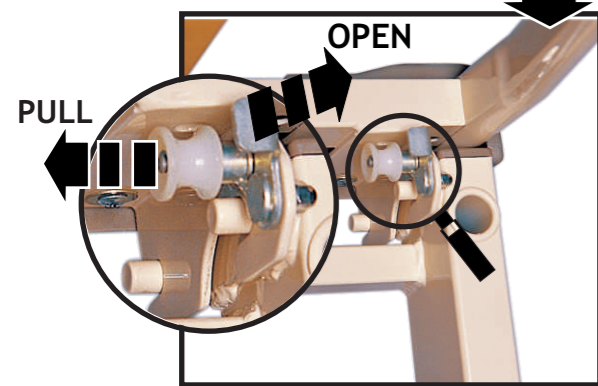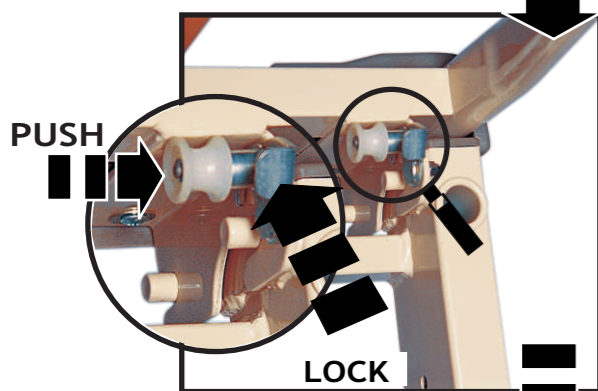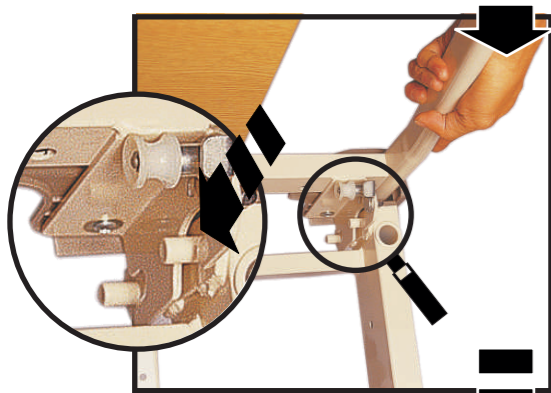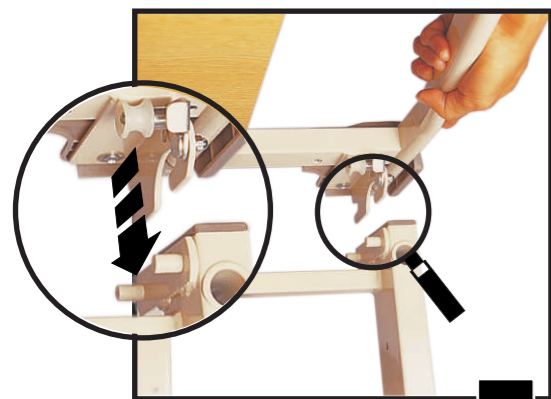Pemasangan Folding Side Rail

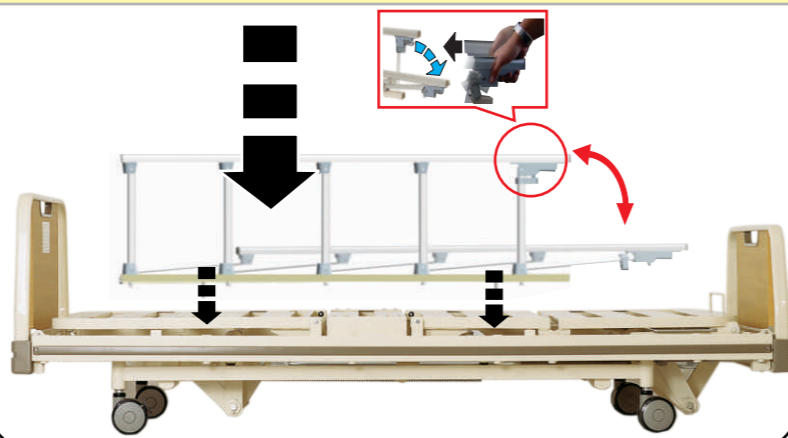
Sistem Crank

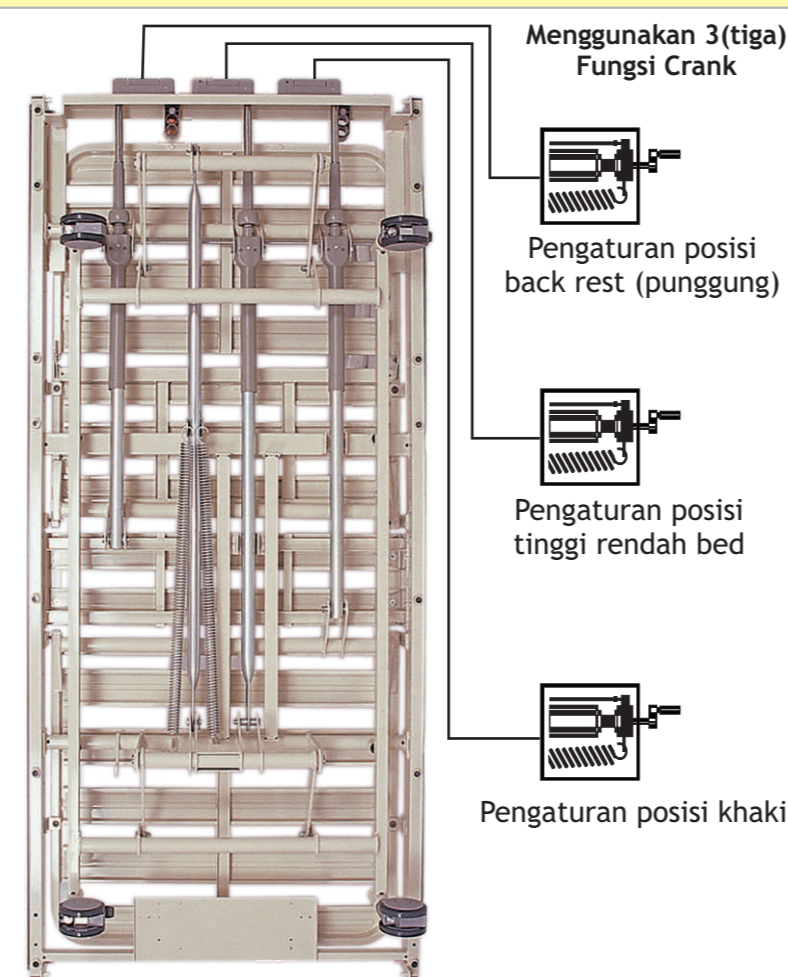Manufactured by :
PT. Chitose Internasional Tbk

Chitose[®]
INNOVATION BY YOUR INSPIRATION

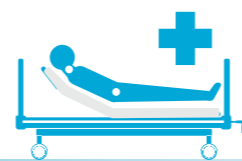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